

- Fast and effective
- Absorbs 45 times of its own weight
- Saves valuable surgical and anaesthetic time.

SURGISPON®

SURGISPON® is a sterile haemostatic absorbable gelatin sponge made from highly purified pharmaceutical grade gelatin (lyophilized hydrolyzed collagen) foam, which gives it a uniform porosity and reacts neutrally. The dried foam is cut, packaged and sterilised using gamma radiation. SURGISPON® is immediately available for use in the operating theatre and does not require any sterilisation by dry heat or autoclaving.

SURGISPON® may be used in combination with antibiotics, thrombin, chemotherapeutics without reduction of the sponges haemostatic effect. Carefully read applicable instructions for use (IFU) before application.



WHERE to use SURGISPON® Dental

To achieve immediate hemostasis in:- Extractions, Oral bone and soft tissue surgery



TOOTH EXTRACTIONS
Oozing bleeding from alveolus is stopped by application of SURGISPON® Dental cube



ENUCLEATION OF CYSTS
Bleeding from the cavity is controlled with the Dental cube



DETAIL
Dental cube place in cavity



CONTROL OF PROFUSE BLEEDING
In tooth extraction by means of several dental cubes

Product ID	Product Name	Dimensions (mm)	No. of Pieces Per Box
SSP-101010.12	CUBES	10X10X10	12
SSP-101010.32	CUBES	10X10X10	32
SSP-101010.50	CUBES	10X10X10	50 Per Jar
SSP-1208	DENTAL TAMPON	12X08Ø	32
SSP-202007	SHEET	20X20X07	20