

VENOME FILLER LIDOCAINE

VENOME FILLER LIDOCAINE

Venome Filler Lidocaine is a range of five tissue fillers, containing monophasic, non-animal, cross-linked hyaluronic acid obtained using bio-fermentation technology. Each Venome product consists of a different amount of sodium hyaluronate and a cross-linking agent, which allows you to find the product perfectly matching both the indication and the desired scope of correction. Whether your goal is to hydrate and revitalize skin, eliminate fine or medium wrinkles, add volume, reshape parts of the face or enhance a client's lips — the Venome products are just what you are looking for


Venome Filler Lidocaine products are based on **innovative TOUCH technology**, which is characterized by its application in the production of stabilized hyaluronic acid o high molecular weight and small deviations between sizes, a properly prepared, physiological solution buffer that hydrates the formula to form a three-dimensional network acid and the addition of 3 mg lidocaine.

The initial technological process of making the Venome Filler Lidocaine involves double purification of basic components to get rid of any cell wall residues, making it possible to obtain HA with very low protein and endotoxin levels (as demonstrated in studies) and minimize the risk of any allergic reactions. The purified product is then evenly distributed in the whole mixture.

The sodium hyaluronate is stabilized using the BDDE cross-linking agent. The goal is to link the molecules by chemical bonds, which will slow down product degradation and provide the necessary resilience to hyaluronidase, an enzyme naturally found in the body. Any excessive BDDE is removed in the process of leaching. This largely determines the structure of the cross-linked HA and minimizes the risk of any unwanted overcorrection during the treatment. The next step involves combining the cross-linked formula and the physiological buffer solution. In this process, the stabilized chains are covered by an aqueous layer, which prevents over-absorption of water during and after the injection, and thus reduces the swelling.

The process of **double homogenization** allows for obtaining a perfectly smooth and consistent gel with high structural viscosity.

The use of the **TOUCH technology** helps obtain uniform and elastic hyaluronic acid, increasing the comfort of treatment with **Venome Filler Lidocaine** products. A fully **hydrated HA formula minimizes the risk of swelling**, making the process of achieving the ideal result easier for the practitioner and helping to administer the right amount of the product, and accelerates recovery.

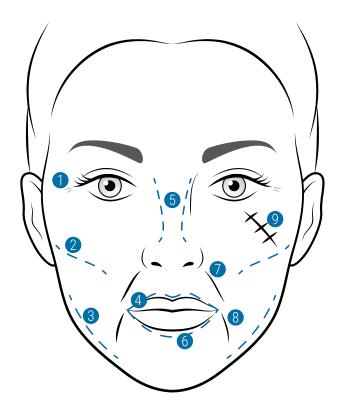

- Fine lines
- VENOME FILLER S ← Lidocaine VENOME FILLER S Lidocaine
- 2 Cheekbones

VENOME FILLER L Lidocaine

- 3 Jawline contouring
 VENOME FILLER L Lidocaine
- 4 Subtle lip enhancement

 VENOME FILLER S⇔ Lidocaine

 VENOME FILLER S Lidocaine
- Nasal reshaping
 VENOME FILLER L Lidocaine

Full lips

VENOME FILLER L ⇔ Lidocaine

VENOME FILLER M Lidocaine

Nasolabial folds

VENOME FILLER S Lidocaine VENOME FILLER M Lidocaine VENOME FILLER L ⇔ Lidocaine VENOME FILLER L Lidocaine

- 8 Marionette lines
 VENOME FILLER S Lidocaine
- 9 Scars and atrophies
 VENOME FILLER M Lidocaine
 VENOME FILLER L Lidocaine

VENOME FILLER S⇔Lidocaine

HYDRATES AND SUBTLY ENHANCES LIPS

INDICATION

- Soft and enhanced, but natural-looking lips
- Delicate contouring
- · Eliminating minor asymmetries
- Removing small perioral lines



INGREDIENTS

The product contains **14 mg/ml of monophasic, cross-linked hyaluronic acid** in the form of viscoelastic gel and 3mg lidocaine hydrochloride.

FFFFCT

Venome Filler S Lips Lidocaine is a biodegradable, non-animal sodium hyaluronate, dedicated to superficial injections aiming at achieving natural results, while also moisturizing the lips. **The product is fully hydrated**, which minimizes the risk of post-treatment swelling. It also makes it easier for the practitioner to fill and contour the lips as it prevents excessive swelling during the injection. After the **Venome Filler S Lips Lidocaine** treatment, lips are soft, smooth and have the right volume. Superficial lines can be subtly filled, without the risk of overcorrection.

The effect lasts between **5** and **6** months, though this depends on the injection technique, as well as a number of biological and environmental factors.

DIRECTIONS FOR USE

Inject in the mucosa or middle layers of the tissue.

Recommended needle and cannula size:

- -27G
- -30G

VENOME FILLER S Lidocaine

REMOVES FINE AND MEDIUM WRINKLES

INDICATION

- Minimizing the appearance of fine and medium wrinkles
- Softening nasolabial folds
- Smoothing marionette lines
- Lip enhancement
- Corner lip lift
- Hand rejuvenation



INGREDIENTS

The product contains **16 mg/ml of monophasic, cross-linked hyaluronic acid** in the form of viscoelastic gel and 3mg lidocaine hydrochloride.

FFFFCT

Venome Filler S Lidocaine is a cross-linked hyaluronic acid gel dedicated to removing fine and medium wrinkles. Owing to the use of **the TOUCH cross-linking technology**, which helps obtain the right elasticity and uniformity, the product is easy to administer. Its plastic properties allow the practitioner to massage the injection site to spread the product evenly across tissues. This ensures natural results and eliminates the risk of overcorrection. **The hydrated formula of Venome Filler S Lidocaine** minimizes the risk of post-treatment swelling, and as a consequence, reduces the recovery time, helping the skin to become firm quicker.

The effect lasts between **5** and **6** months, though this depends on the injection technique, as well as a number of biological and environmental factors.

DIRECTIONS FOR USE

Inject in the mucosa or middle layers of the tissue.

Recommended needle and cannula size:

- -27G
- -30G

Professional use only venome-global.com

VENOME FILLER M Lidocaine

REMOVES MEDIUM AND DEEP LINES, INCREASES SKIN ELASTICITY, ELIMINATES SAGGING

INDICATION

- Removing medium and deep wrinkles
- Smoothing marionette lines
- Treating small scars
- Eliminating nasolabial folds
- Lip augmentation



INGREDIENTS

The product contains **20 mg/ml of monophasic, cross-linked hyaluronic acid** in the form of viscoelastic gel and 3mg lidocaine hydrochloride.

FFFFCT

Venome Filler M Lidocaine is a versatile product for complete removal of medium wrinkles and softening deep wrinkles and nasolabial folds. Owing to its fully hydrated formula and the use of the unique **TOUCH cross-linking technology**, this monophasic hyaluronic acid gel spreads evenly in the tissue, making the skin firm and smooth and eliminating wrinkles without excessive swelling — which can happen when **the HA molecules are overly hygroscopic**. Can be used to complement treatments with Venome Filler S Lidocaine and L Lidocaine.

The effect lasts between **5** and **6** months, though this depends on the injection technique, as well as a number of biological and environmental factors.

DIRECTIONS FOR USE

Inject in the mucosa, middle or deep layers of the tissue.

Recommended needle and cannula size:


- -27G
- -30G

VENOME FILLER L⇔ Lidocaine

PERFECTLY FILLED LIPS WITH A DEFINED CUPID'S BOW

INDICATION

- Increasing lip volume
- Eliminating upper or lower lip asymmetries
- Cupid's bow enhancement
- · Perioral rejuvenation
- · Lifting the drooping corners
- · Rejuvenation of the top of the hand

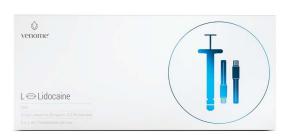

INGREDIENTS

The product contains **20 mg/ml of monophasic, cross-linked hyaluronic acid** in the form of viscoelastic gel and 3mg lidocaine hydrochloride.

EFFECT

Venome Filler L Lips Lidocaine is a cross-linked, hyaluronic acid gel with exceptional rheological properties, perfect for augmenting and enhancing lips. This uniform and homogeneous gel gives the lips the right volume, makes them plump and hydrated, while at the same time keeping them soft and natural. Owing to its viscoelastic structure and the use of the HA hydration technology, the filler can effectively eliminate any upper or lower lip asymmetries, without producing a "fake" look. Venome Filler L Lips Lidocaine is the perfect choice if you want to achieve height, volume, and border definition without excessive forward projection — it ensures the upper lip rolls out nicely and enhances the Cupid's bow. The result — full, sensuous lips.

The effect lasts between **5** and **6** months, though this depends on the injection technique, as well as a number of biological and environmental factors.

DIRECTIONS FOR USE

Inject in the mucosa, middle or deep layers of the tissue.

Recommended needle and cannula size:

- -27G
- -30G

Professional use only venome-global.com

VENOME FILLER L Lidocaine

VOLUME RESTORATION

— SHAPING AND ENHANCING NATURAL BEAUTY

INDICATION

- Face contouring
- Cheekbone and jawline contouring
- Nasal reshaping
- Eliminating nasolabial folds
- Treating scars or atrophies



INGREDIENTS

The product contains **24 mg/ml of monophasic, cross-linked hyaluronic acid** in the form of viscoelastic gel and 3mg lidocaine hydrochloride.

FFFFCT

Venome Filler L Lidocaine is a product dedicated to restoring face volume. It is a monophasic, uniform product which helps **smooth out medium and deep wrinkles**. When aging skin on the cheeks loses volume, it becomes saggy, which makes the nasolabial folds more prominent. The use of **the TOUCH technology** and the optimal method of cross-linking helped obtain a gel of **the perfect density for restoring volume** in the middle part of the face. **Venome Filler L Lidocaine** effectively replaces the lost tissue, contours the face and helps achieve the desired shape.

The effect lasts between **6** and **9** months, though this depends on the injection technique, as well as a number of biological and environmental factors.

DIRECTIONS FOR USE

Inject in the mucosa, middle or deep layers of the tissue.

Recommended needle and cannula size:

- -27G
- -30G


PRODUCT/ PROBLEM	S 😂	S Lidocaine	M Lidocaine	L Control	L Lidocaine
SUBTLE LIP ENHANCEMENT	•				
FINE LINES	•				
NASOLABIAL FOLDS			•		•
MARIONETTE LINES					
FULL LIPS				•	
CHEEKBONES					•
JAWLINE CONTOURING					
NASAL RESHAPING					•
SCARS AND ATROPHIES			•		•
(HANDS		•			•

Professional use only venome-global.com

Venome FILLER



- now also with lidocaine!

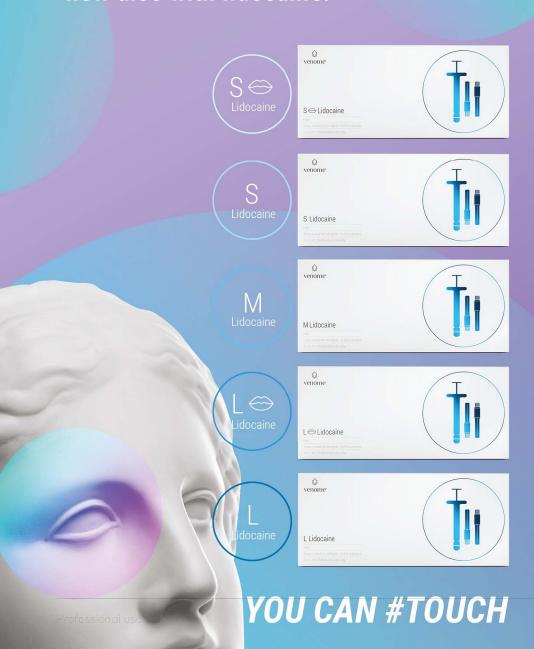

POST-TREATMENT RECOMMENDATIONS

WITHIN 48 HOURS AFTER THE TREATMENT:

\bigcirc	drink plenty of water (about 2.5 liters a day),
\bigcirc	keep injection sites particularly clean (do not touch),
\bigcirc	limit contact with animals (touching, hugging),
\bigcirc	do not sleep in the prone position,
\bigcirc	do not drink alcoholic and caffeinated drinks,
\bigcirc	do not smoke tobacco and cigarettes,
\bigcirc	do not apply makeup,
\bigcirc	give up using the sauna, swimming pool, hot baths, peels, depilation,
	intense exercise,
\bigcirc	do not apply preparations with an irritating effect on the skin,
\bigcirc	use protective creams with a UV filter, min. SPF 50,
\bigcirc	use strongly regenerating and moisturizing creams,
\bigcirc	do not use other aesthetic treatments on irritated areas.



office@venome-global.com

②venomeofficial

www.venome-global.com